

* Warequip accepts no responsibility for equipment that is operated incorrectly, or not in accordance with operation manuals.

Wheelchair Mover



250 KG

LOAD CAPACITY – 250kg patient plus chair on a level surface

Ensure patient is seated at all times as recommended
DO NOT OVERLOAD or obstruct operator view with high loads.



OPERATION

The unit is designed to be operated in front of the user. The front of the unit is the hitch end.



POWER – ON/OFF SWITCH

The On/Off switch is located on the control panel, switch to on position for normal operation.
If powered status light flashes there may be a fault, please contact your supplier.



EMERGENCY STOP BACK OFF BUTTON

Stops the unit if activated when reversing.



FAST /SLOW MODE

The switch is designed to limit the maximum available speed. Push the switch to get either the rabbit or the turtle speed. The rabbit is a fast walking pace and the turtle is a slow walking pace. It is recommended Turtle mode is used when hitching or for use in tight confined spaces.



PADDLE LEVER

When facing the machine; (Forward motion is to the hitch end)
Push right hand lever forward for forward motion.
Push left hand lever forward for reverse motion.
The speed is determined by how far you push the paddle.
The Wheelchair should be in front of you when walking.



BATTERY INDICATOR

When fully charged all lights will illuminate to show 100%, as you use the tug the lights will go off to show the level. Once the charge falls to approximately 40% we recommend charging, the last red light on the fuel gauge will flash for approximately 10 minutes before the unit loses all charge.

Wheelchair Mover



CHARGING


The tug has an internal charger which is supplied within the machine. A separate charging lead has been supplied on the back of the unit.

The charging point is located on the back left hand side of the body.

Plug the charger lead into the plug on the tug (the plug will only fit the correct end) and then into power point.
– Charge with On/Off switch in the OFF POSITION.

OVERNIGHT CHARGE IS RECOMMENDED

ONLY USE THE CHARGER LEAD SUPPLIED WITH THE UNIT.

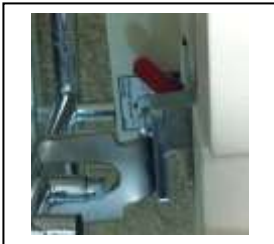

BRAKE OVERRIDE

In the rare event of failure or if the batteries have no charge you can override the park brake. Located under the hitch on the right hand side you will notice a red lever, simply push the lever forward to release the brakes. Only use this function on a level surface. To reengage pull the lever back into position.



HITCHING

Various hitches can be supplied with the unit to suit your application. Always ensure connection takes place on a level surface.

Activate the brakes on the Wheelchair. Approach the WC at a slow speed, ensure the centre of the hitch is in line with the centre of the WC. If the hitch is not centred you can use the foot pedals to adjust the position.

To activate the hitch simply push down on the black lever on the back upper right side on the mover, push down until connection is made, the hitch will automatically lock into position. To release hold onto the black lever and silver lever underneath, pull the lever up.

Always ensure the Wheelchair mover has made correct connection before unlocking the WC brake and driving away.



Please ensure you are familiar with the manual and the correct operating instructions.

If in doubt contact supplier before use.